



Setting up the O2-RESQ System

- 1** Connect Generator directly to a 50psi Oxygen source on a Flow Meter or to a DISS fitting on an O2 tank regulator



Remove the mask from Inner bag and attach the circuit to the mask. Turn on oxygen. Hand mask to patient for them to place on their face to get used to the flow. Then proceed to put the Head Strap on the patient.



For all mask styles, Flip Head Strap forward and place mask on the patient's face. If using the BiTrac ED Mask, pinch the OmniClip, slide it up or down to find the best position on patient's forehead



For all mask styles, Flip Head Strap back over the patient's head, bring tabs forward on the top Head Strap and adjust equally to proper fit. Fold straps back to attach. Next attach the bottom 2 clips and repeat above sequence



Finally if using the BiTrac ED Mask, adjust the OmniClip in and out on Head Strap and Mask for best fit: Do Not Over Tighten the Head Strap. Monitor the Patient.